PROVOX[®]

FreeHands FlexiVoice[™]


Rx
ONLY

42° C
108° F
RT
2° C
36° F

Figure 1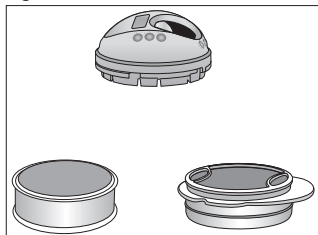

Provox FreeHands
HME

Provox Life™
FreeHands HME

Figure 2

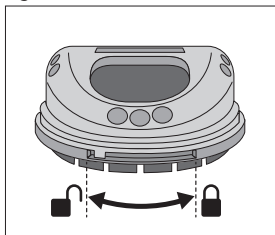
Figure 3

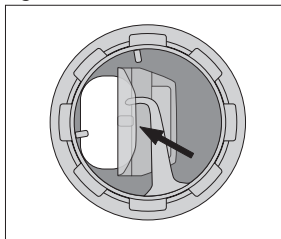
Figure 4

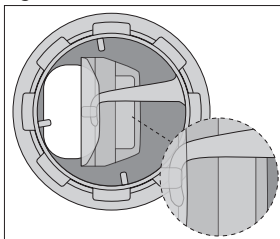
Figure 5

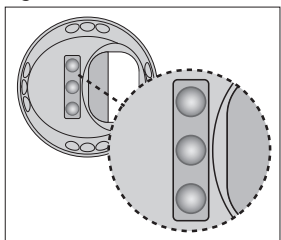
Figure 6

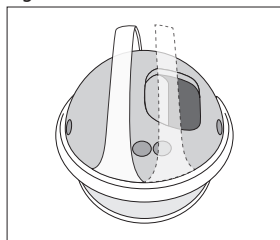
Figure 7

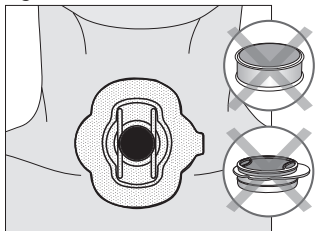
Figure 8

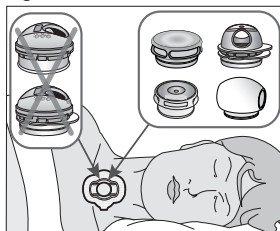
Figure 9

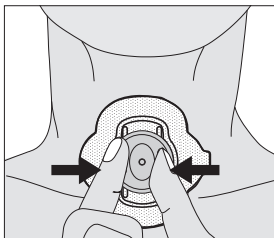
Figure 10

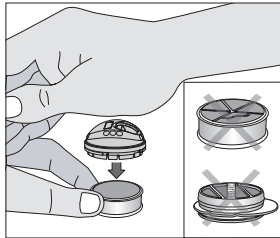
Figure 11

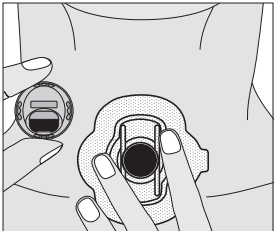
Figure 12

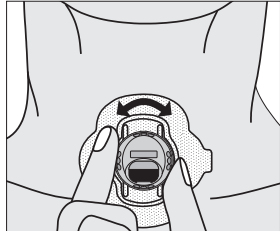
Figure 13

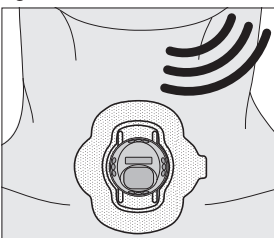
Figure 14

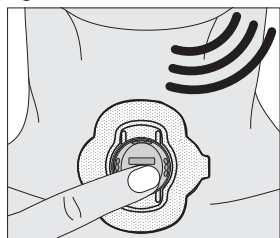
Figure 15

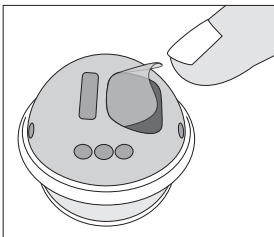
Figure 16

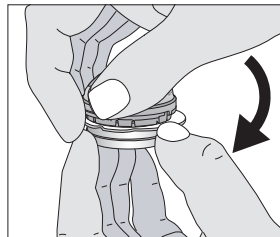
Figure 17

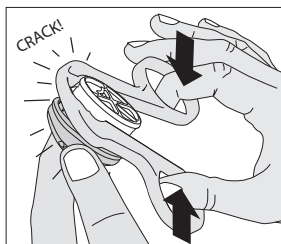
Figure 18

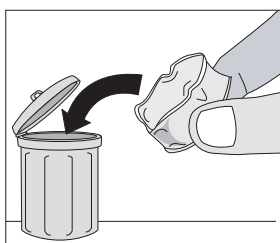
Figure 19

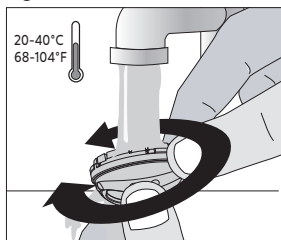
Figure 20

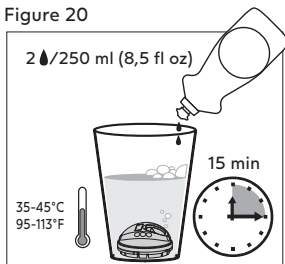
Figure 21

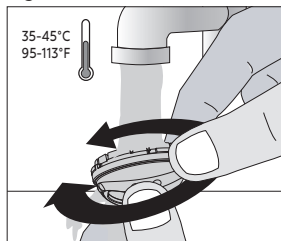
Figure 22


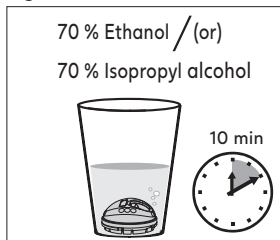
Figure 23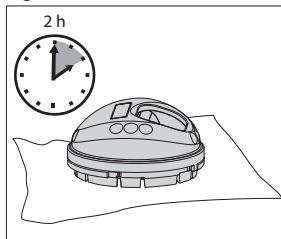

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. FreeHands FlexiVoice™ and Provox® Life™ are trademarks of Atos Medical AB. For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore; Fabricante; Fabricante; Tillverkare; Producent; Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător; Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici; მწარმოებელი; Ἀρτοποιητής; İstehsalçı; Производитель; Produsen; Pengilang; 製造業者; 제조사; 製造商; 制造商; 製造商; היצרן; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de fabrication; Data di fabbricazione; Fecha de fabricación; Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev; Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás időpontja; Dátum výroby; Datum proizvodnje; Data produkcji; Data fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία κατασκευής; Дата на производство; Üretim tarihi; წარმოების თარიღი; Ἀρτοποιημάτων ἀντιστοιχία; İstehsal tarixi; Дата изготовления; Tarikh pembuatan; 製造日; 제조일자; 製造日期; 製造日期; تاریخ التصنيع; תאריך ייצור

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite d'utilisation; Data di scadenza; Fecha de caducidad; Data de validade; Används senast; Anvendes inden; Brukes innen; Viimeinen käyttöpäivä; Síðasti notkunardagur; Kööblik kuni; Derīguma termiņš; Naudoti iki; Datum expirace; Felhasználatóság dátuma; Dátum najneskoršej spotreby; Rok uporabnosti; Data ważności; Data expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок на годност; Son kullanım tarihi; Յարգօժօժ; Ἐπιχρηστέα ἡμερομηνία; Son istifadə tarixi; Срок годности; Gunakan sebelum tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期; 使用截止日期; تاریخ انتهاء الصلاحية; תאריך אחרון לשימוש

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1. Descriptive information

1.1 Intended use

The Provox FreeHands FlexiVoice combines pulmonary rehabilitation using a Heat and Moisture Exchanger with voice rehabilitation using an automatic speaking valve or manual occlusion, in laryngectomized patients using a voice prosthesis.

1.2 CONTRAINDICATIONS

The product shall not be used by patients with a decreased level of consciousness, patients with reduced mobility of the arms and/or hands, or patients who are unable to remove the device themselves.

1.3 Description of the devices

Provox FreeHands FlexiVoice consists of two parts assembled together, a speaking valve for single patient use and a disposable HME cassette (Fig. 1).

The speaking valve has two modes; automatic speaking mode and locked mode (Fig. 2). Rotating the top of the speaking valve moves the device into the automatic speaking or the locked position. Manual occlusion is possible in both modes.

The speaking valve is in automatic speaking mode when the flexible membrane is not locked by the hook (Fig. 3). In automatic speaking mode, the membrane can be closed automatically by the exhaled air. The air is then redirected through the voice prosthesis. The membrane opens again automatically when breathing in.

In locked mode, the flexible membrane is locked by a hook that goes into a loop in the membrane (Fig. 4). This prevents the membrane from closing when breathing heavily, for example, during physical activity.

There are four different speaking valves, each with a different strength membrane: light, medium, strong and xtrastrong.

The number of dots represents the strength of the membrane (Fig. 5).

1 dot = Light membrane (easiest to close)

2 dots = Medium membrane

3 dots = Strong membrane

4 dots = XtraStrong membrane

Provox FreeHands FlexiVoice Arch is an accessory that can be used with the Provox FreeHands FlexiVoice to keep fabric away from the opening of the device. It is simply pressed onto the front of the Provox FreeHands FlexiVoice (Fig. 6). Please note that in some positions the Arch may restrict the possibility of manual occlusion.

The Removal Aid (Fig. 17) can be used to remove the Provox FreeHands HME cassette from the speaking valve.

1.4 Technical data

	Pressure drop at 30 l/min*(Pa)	Pressure drop at 60 l/min*(Pa)	Pressure drop at 90 l/min* (Pa)	Moisture loss at VT=1000ml* (mg/l)
FlexiVoice & FreeHands HME Moist	70	220	490	24
FlexiVoice & FreeHands HME Flow	65	210	470	25
FlexiVoice & FreeHands Life™ HME	56	200	450	24

HME internal volume (Dead space)* max 7 ml

* According to ISO 9360

1.5 WARNINGS

- Do not use the HME cassette without assembling it first with Provox FreeHands FlexiVoice Speaking Valve or HME DigiTop. The HME can get stuck in the attachment holder when used by itself (Fig. 7).
- Do not use Provox FreeHands FlexiVoice with any other attachment devices than original Provox HME attachment devices. The product can get stuck in the attachment device. The attachment device then has to be removed or may be damaged, possibly leading to restricted breathing.
- Do not reuse any part of the Provox FreeHands FlexiVoice between patients. This may cause cross contamination. The device is intended for single patient use only.

1.6 PRECAUTIONS

- Do not modify the product. The speaking function and/or HME function will be lost. Consult your clinician if there is a problem with the device.
- Always clean the speaking valve according to the instructions below. If the product is not properly cleaned, it may cause infection and product malfunction.
- Do not administer medicated nebulizer treatment over the device since the medication can become deposited in the device.
- Do not use humidifiers or heated humidified oxygen via a mask over the tracheostoma while using the device. The HME will become too wet. If oxygen therapy is required, use only non-heated humidified oxygen.

2. Operating instruction

Always test the function of the device prior to use. Check that the device is undamaged and no restricted breathing. Do not use the speaking valve during sleep. The opening of the speaking valve may be covered or the speaking valve may unintentionally be turned to speaking mode resulting in restricted breathing. During sleep, use an HME or an HME DigiTop assembled with an HME cassette (Fig. 8).

Membrane strength selection

Try the Medium membrane (2 dots) first. If you experience unintended closing of this membrane during normal breathing or light activity, try the Strong or the XtraStrong membrane (3 or 4 dots). If you feel it takes too much effort to close the Medium membrane for speaking, try the Light membrane (1 dot). Make sure the membrane does not close unintentionally during regular breathing. You can also use a stronger membrane when you are more active and a lighter one when you are less active.

Assembly

Place the HME cassette on a flat surface the open foam surface up (Fig. 10). Place the speaking valve on top of the HME cassette and push it straight down until it clicks in place. Check that the speaking valve is firmly attached to the HME cassette.

Insertion and usage

Insert the Provox FreeHands FlexiVoice into the connector of the Provox HME attachment or Provox Life™ attachment (Fig. 11). Breathe normally. Switch the device between the modes by rotating the top of the speaking valve until it stops in each position (Fig. 12).

Caution: Rotate only the top of the speaking valve when changing modes, not the entire device. Rotating the entire device can cause uncomfortable movement of the attachment device.

Automatic speaking mode

Provox FreeHands FlexiVoice is in automatic speaking mode when the flexible membrane can move freely and is not locked by the hook (Fig. 3). In this mode, the increase in exhaled airflow for speaking automatically closes the membrane (Fig. 13).

Manual occlusion

Provox FreeHands FlexiVoice can also be manually occluded for speaking by occluding the front opening with a finger (Fig. 14). Manual occlusion can be used in both automatic speaking mode and in locked mode. HME DigiTop can be manually occluded for speaking by occluding the front opening with two fingers (Fig. 9).

Coughing

In automatic speaking mode, the membrane can pop out when coughing. If this happens, breathe normally and push the membrane back through the opening with your fingertip (Fig. 15).

Removal of the HME cassette from the speaking valve

The HME cassette is intended for single use. It should be removed at least at the end of the day or earlier if needed.

Caution: Do not attempt to wash the HME cassette since this will wash away the impregnation essential for its function.

Remove the Provox FreeHands HME with the Removal aid (Fig. 17) or Provox Life™ FreeHands HME with your fingers according to Fig 16. Discard the HME cassette (Fig 18).

Cleaning

In order to keep the speaking valve and HME DigiTop (speaking device) in a good working condition and to avoid build-up of microorganisms, it should be cleaned and disinfected daily.

Caution: Do not clean or disinfect the speaking device in any other way than described below. Do NOT boil the device and do NOT use hydrogen peroxide

for disinfection since these actions will damage the device. Clean the speaking valve according to Fig 19 to 21 and 23.

1. Remove the HME cassette from the speaking device as described above.
2. Rinse the speaking device carefully on both sides under lukewarm (20-40 °C / 68-104 °F) drinking water while rotating the lid back and forth for 2 minutes (Fig. 19).
3. Mix 2 drops of dish soap in 250ml warm (35-45 °C / 95-113 °F) drinking water. Do not use boiling water since this may damage the speaking device.
4. Put the speaking device in speaking mode and place the speaking device in the mixture for 15 minutes (Fig. 20).
5. Rinse the speaking device on both sides in warm (35-45 °C / 95-113 °F) drinking water for 15 seconds (Fig. 21). In order to reach all parts of the speaking device, be sure that it is switched between speaking and locked mode a couple of times during rinsing.
6. Inspect the device regarding mucus and debris residues. Repeat the steps above if necessary.
7. After cleaning, place the speaking device in speaking mode on clean gauze and leave it to air-dry for at least 2 hours (Fig. 23). When the speaking device is completely dry, it can either be used again or be placed in a clean, closed container for storage.

Disinfection

Disinfection might be needed if the speaking device looks dirty or there has been a risk of contamination. A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination. Then the device should be both cleaned and disinfected before use. If disinfection is needed first clean the device as described above and then disinfect by using the following procedure:

1. First follow the cleaning procedure as described in section “Cleaning” above, without step 7. After cleaning, put the speaking device in speaking mode and submerge it for 10 minutes either in ethanol 70% or isopropyl alcohol 70% (Fig. 22).

Caution: Do NOT use hydrogen peroxide.

2. After cleaning and disinfection, put the speaking device on a clean gauze and leave it to air-dry for at least 2 hours (Fig. 23). Make sure that the speaking valve is in speaking mode.
3. When the speaking device is completely dry, it can either be used again or placed in a clean, closed container for storage.

Caution: Do not use the speaking device until it is completely dry. Inhalation of disinfectant fumes can cause severe coughing and irritation of the airways.

2.1 Device lifetime

Provox FreeHands FlexiVoice speaking valve can be used for a maximum of 6 months as long as it is intact and functioning as intended.

The HME cassette is a single use device and must be replaced at least every 24 hours, or more often if needed.

2.2 Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

2.3 Compatible products

Caution: Use only original components. Other accessories may cause product malfunction.

Provox FreeHands HME are compatible with Provox HME attachment (Provox Adhesives, Provox LaryTubes and Provox LaryButtons).

Provox Life™ FreeHands HME are compatible with Provox Life™ attachment (Provox Life™ Adhesives, Provox Life™ LaryTubes and Provox Life™ LaryButtons).

HME DigiTop enable use of HME cassette without the speaking valve, and can manually be occluded to enable speaking. For single patient use with a device life time of 2 months.

Removal Aid can be used to help to remove Provox FreeHands HME from the speaking valve.

Provox FreeHands FlexiVoice Arch is an accessory that can be used with the Provox FreeHands FlexiVoice to keep fabric away from the opening of the device.

3. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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